
BUBBLES THE FISH™ II PEDIATRIC AEROSOL MASK

INSTRUCTIONS FOR USE

For single patient use (one person) only. For use with PARI Nebulizers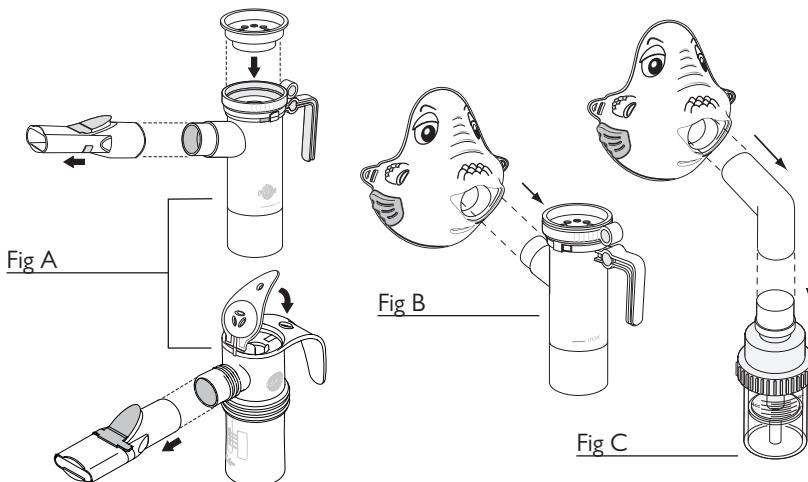

!WARNING

READ ALL WARNINGS AND INSTRUCTIONS BEFORE USE. IMPROPER USE CAN RESULT IN SERIOUS OR FATAL ILLNESS OR INJURY.

The enclosed elbow adapter should be used only with a "T" piece nebulizer and the Bubbles the Fish™ II pediatric aerosol mask. Do not use elbow adapter with a PARI nebulizer or any other masks

PARI's Bubbles the Fish™ II pediatric aerosol mask is designed with Pro-Vent™ technology. This optimizes inhalation of respirable particles to the lungs while dramatically reducing facial & ocular deposition, resulting in safer treatments.

PLEASE FOLLOW STEP-BY-STEP AND REFER TO ILLUSTRATIONS ABOVE.

1. *With PARI Nebulizer:* Remove the mouthpiece from the nebulizer and ensure inspiratory valve cap is attached (Fig A). Then press mask firmly onto the nebulizer outlet (Fig B).
2. *With Traditional "T" nebulizer:* Attach included elbow adapter to nebulizer, then attach mask firmly to the elbow piece (Fig C).
3. *Elastic Strap:* Your mask includes an elastic strap which can be used for increased stability during treatments. If this is not needed, simply remove it from the mask and store in a safe place.
4. *Cleaning:* Wash with warm water and liquid dish soap. Rinse thoroughly then allow to dry on a clean, lint-free cloth and store. Do not place mask in dishwasher or autoclave.

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© 2010 PARI Respiratory Equipment, Inc.
PARI Respiratory Equipment, Inc.